

PL475

Patient Lift Scale

Owner's Manual

0046-M496-O1 Rev A 11/99 CARDINAL SCALE MFG. CO. PO BOX 151 • WEBB CITY, MO 64870 PH 417-673-4631 • FAX 417-673-5001

Printed in USA

Thank you for purchasing our DETECTO[™] Model PL475 Patient Lift Scale. It has been manufactured with quality and reliability at our factory in Webb City, MO USA. This scale has been tested before leaving our factory to insure accuracy and dependability for years to come.

When removing the unit from its packing, inspect it for signs of damage, such as exterior dents and scratches. Keep the carton and packing material for return shipment if it should become necessary. It is the responsibility of the purchaser to file all claims for any damages or loss incurred during transit, unless this responsibility has been accepted by the seller in its proposal.

This manual is provided to guide you through operation and maintenance of your Patient Lift Scale. Please read it thoroughly before attempting to install or operate your PL475 and keep it handy for future reference.

FCC COMPLIANCE STATEMENT WARNING!

This equipment generates, uses and can radiate radio frequency and if not installed and used in accordance with the instruction manual, may cause interference to radio communications. It has been tested and found to comply with the limits for a Class A computing device pursuant to Subpart J of Part 15 of FCC rules, which are designed to provide reasonable protection against such interference when operated in a commercial environment. Operation of this equipment in a residential area may cause interference in which case the user will be responsible to take whatever measures necessary to correct the interference.

All rights reserved. Reproduction or use, without expressed written permission, of editorial or pictorial content, in any manner, is prohibited. No patent liability is assumed with respect to the use of the information contained herein. While every precaution has been taken in the preparation of this manual, the Seller assumes no responsibility for errors or omissions. Neither is any liability assumed for damages resulting from use of the information contained herein. All instructions and diagrams have been checked for accuracy and ease of application; however, success and safety in working with tools depend to a great extent upon the individual accuracy, skill and caution. For this reason the Seller is not able to guarantee the result of any procedure contained herein. Nor can they assume responsibility for any damage to property or injury to persons occasioned from the procedures. Persons engaging the procedures do so entirely at their own risk.

TABLE OF CONTENTS

Specifications 1
Introduction 2
Setup Instructions 2
Transporting The Patient
Weighing The Patient 4
Component Identification
Preventive Maintenance6
Service Information 6
Service Hints 6
Parts Identification7

SPECIFICATIONS

INDICATOR

Capacity:	400 lbs (180 kg)
Resolution:	0.2 lb (0.1kg)
Display:	LCD digital display
Power Requirements:	9.0 V Battery (<i>included</i>)
Temperature Range:	50° to 104° F (+10° to +40° C)

<u>LIFT</u>

Casters:	Four (4); full swivel, rear pair locking
Lift Mechanism:	Hydraulic
Closed Overall Dimensions:	54"H x 22"W x 50"L (1.37m H x 0.56m W x 1.27m L)
Under bed clearance: Shipping Weight:	4.8"H (122mm) 250 lbs (113kg)

SERIAL NUMBER _____ DATE OF PURCHASE _____ PURCHASED FROM _____

RETAIN THIS INFORMATION FOR FUTURE USE

PRECAUTIONS

Before using this instrument, read this manual and pay special attention to all "WARNING" symbols:



This is a "Table of Contents preview" for quality assurance

The full manual can be found at http://the-checkout-tech.com/estore/catalog/

We also offer free downloads, a free keyboard layout designer, cable diagrams, free help and support.

<u>http://the-checkout-tech.com</u> : the biggest supplier of cash register and scale manuals on the net